



Sinu-Knit®

Sinu-Knit Dissolvable Post-op Surgical Dressing

Sinu-Knit is the only dissolvable carboxymethylcellulose (CMC) dressing that provides a moist hydrocolloid physical barrier and minimizes post-op adhesions and bleeding.

Sinu-Knit Conversion



Dry Sinu-Knit Dressing

Partially Hydrated Sinu-Knit Dressing

Sinu-Knit Gel



Sinu-Knit Intraoperative

Provides a Moist Physical Barrier

- Intended to support and separate mucosal tissue by providing a moist physical barrier
- Minimizes post-op adhesions

Minimizes Bleeding

- Minimizes intra- and postoperative oozing

Dissolves Away

- Dissolves away through normal outflow in 5-7 days
- Easy post-op debridement
- No need to remove post-op packing

Catalog No.	Nasal/Sinus Solution	Units Per Box
RR 600	Sinu-Knit Dissolvable Dressing	12

Sinu-Knit Dissolvable Dressing: Technique Guide



Insertable Sinu-Knit Dressing



Unique handling—easily placed in sinus cavity



Wet Sinu-Knit fabric with sterile water



Hydrated Sinu-Knit converts to dense gel stent

- 1 Using dry forceps, place the Sinu-Knit into the surgical site.
NOTE: Wet forceps will initiate gelling, making it sticky and difficult to place.
- 2 Fold Sinu-Knit into a “>” shape with forceps on outside and place into surgical site.
- 3 Use **Sterile Water only** to hydrate Sinu-Knit.
NOTE: Saline may inhibit the Sinu-Knit conversion to a dense gel and may not fully activate its hemostatic properties.
- 4 Use a minimum 10cc sterile water per side to hydrate Sinu-Knit completely, working from posterior to anterior, to ensure that **ALL** fibers are fully saturated.
- 5 Do not apply lubricants or antibiotic ointments to the Sinu-Knit fabric. They will inhibit the hydrocolloid and hemostatic properties of the material.
- 6 Patient should irrigate with isotonic saline spray daily until first office visit. Sinu-Knit dissolves via normal sinus outflow in 5-7 days.
NOTE: Patient should avoid aggressive bulb-type flushing during first week after surgery.



ArthroCare ENT
680 Vaqueros Avenue
Sunnyvale, CA 94085-3523
main phone 408-736-0224
order entry **800-797-6520**
order entry fax 888-994-2782
www.arthrocareENT.com

ArthroCare Europe AB
Skeppsbron 2
111 30 Stockholm
Sweden
phone +46 8 546 172 00
fax +46 8 546 172 39
info@arthrocare.se

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.

© 2010 ArthroCare Corporation. ArthroCare and Sinu-Knit are trademarks and/or registered trademarks of ArthroCare Corporation.

P/N 33591 RevA